User Guide

Giant

Extra Loud Speakerphone

IMPORTANT SAFETY INSTRUCTIONS

To prevent severe injury and loss of life/property, read this section carefully before using the product to ensure proper and safe operation of your product.

Power connection

- Use only the power source marked on the product.
- Do not overload power outlets and extension cords. This can result in the risk of fire or electric shock.
- Completely insert the AC adapter/power plug into the power outlet. Failure to do so may cause electric shock and/or excessive heat resulting in a fire.
- Regularly remove any dust, etc. from the AC adapter/power plug by pulling it from the power outlet, then wiping with a dry cloth. Accumulated dust may cause an insulation defect from moisture, etc. resulting in a fire.
- Unplug the product from power outlets if it emits smoke, and abnormal smell, or makes an unusual noise. These conditions can cause fire or electric shock. Confirm that smoke has stopped emitting and contact the Clarity Customer Service at 800-426-3738.
- Unplug from power outlets and never touch the inside of the product if its casing has been broken open.
- Never touch the plug with wet hands. Danger of electric shock exists.

Installation

- To prevent the risk of fire or electrical shock, do not expose the product to rain or any type of moisture.
- Do not place or use this product near automatically controlled devices such as automatic doors and fire alarms. Radio waves emitted from this product may cause such devices to malfunction resulting in an accident.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

- Do not allow the AC adaptor or telephone line cord to be excessively pulled, bent or placed under heavy objects.

Operating safeguards

- Unplug the product from power outlets before cleaning. Do not use liquid or aerosol cleaners.
- Do not disassemble the product.
- Do not spill liquids (detergents, cleansers, etc.) onto the telephone line cord plug, or allow it to become wet at all. This may cause a fire. If the telephone line cord plug becomes wet, immediately pull it from the telephone wall jack, and do not use.

Medical

- Consult the manufacturer of any personal medical devices, such as pacemakers or hearing aids, to determine if they are adequately shielded from external RF (radio frequency) energy. (The product operates in the frequency range of 1.92 GHz to 1.93 GHz, and the RF transmission power is 115 mW (max.). Do not disassemble the product.
- Do not use the product in health care facilities if any regulations posted in the area instruct you not to do so. Hospitals or health care facilities may be using equipment that could be sensitive to external RF energy.

SAVE THESE INSTRUCTIONS

IMPORTANT BATTERY SAFETY INSTRUCTIONS

- Do not mix old and new batteries.
- Do not open or mutilate the batteries. Released electrolyte from the batteries is corrosive and may cause burns or injury to the eyes and skin. The electrolyte is toxic and may be harmful if swallowed.
- Exercise care when handling the batteries. Do not allow conductive materials such as rings, bracelets, or keys to touch the batteries, otherwise a short circuit may cause the batteries and/or the conductive material to overheat and cause burns.
- Charge the batteries (optional) identified for use with this product only, in accordance with the instructions and limitations specified in this manual.
- Only use a compatible base unit (or charger) to charge the batteries. Do not tamper with the unit (or charger). Failure to follow these instructions may cause the batteries to swell or explode.

The base batteries are optional (for backup only). If you choose to install backup batteries, **use rechargeable batteries ONLY**. The backup batteries should be AA, NiMH, rechargeable only, rated 1600mAh. They must be fully charged for 8 hours before using the backup batteries (before using the phone without the power adapter, during a power outage).

The backup batteries ensure the functionality of your unit for 3-4 hours in case of a power outage.

To install batteries in the base:

1. Turn the unit over and open battery compartment door.
2. Install batteries according to the polarity shown inside the battery compartment.
3. Close the battery compartment.

**WARNING: USE RECHARGEABLE BATTERIES ONLY.
DO NOT USE ALKALINE BATTERIES!**

**CAUTION: RISK OF EXPLOSION IF BATTERY IS REPLACED
BY INCORRECT TYPE. DISPOSE OF USED BATTERIES
ACCORDING TO THE INSTRUCTIONS.**

SAVE THESE INSTRUCTIONS

CARE AND MAINTENANCE

FOR BEST PERFORMANCE

Base unit location/avoiding noise

The base unit and other compatible Clarity units use radio waves to communicate with each other.

- For maximum coverage and noise-free communications, place your base unit at a convenient, high, and central location with no obstructions between the handset and base unit in an indoor environment.
 - a. away from electronic appliances such as TVs, radios, personal computers, wireless devices, or other phones.
 - b. facing away from radio frequency transmitters, such as external antennas of mobile phone cell stations. (Avoid putting the base unit on a bay window or near a window.)
- Coverage and voice quality depends on the local environmental conditions.
- If the reception for a base unit location is not satisfactory, move the base unit to another location for better reception.

Environment

- Keep the product away from electrical noise generating devices, such as fluorescent lamps and motors.
- The product should be kept free from

excessive smoke, dust, high temperature, and vibration.

- The product should not be exposed to direct sunlight.
- Do not place heavy objects on top of the product.
- When you leave the product unused for along period of time, unplug the product from the power outlet.
- The product should be kept away from heat sources such as heaters, kitchen stoves, etc. It should not be placed in rooms where the temperature is less than 0 °C (32 °F) or greater than 40 °C (104 °F). Damp basements should also be avoided.
- The maximum calling distance may be shortened when the product is used in the following places: Near obstacles such as hills, tunnels, underground, near metal objects such as wire fences, etc.
- Operating the product near electrical appliances may cause interference. Move away from the electrical appliances.

Routine care

- Wipe the outer surface of the product with a soft moist cloth.
- Do not use benzene, thinner, or any abrasive powder.

TECHNICAL SPECIFICATIONS

Dimensions:

10.197 in. x 6.299 in. x 5.118 in.

Weight: 32.275 oz. (without AC adapter)

Standard: DECT 6.0 (Digital Enhanced Cordless Telecommunications 6.0)
Bluetooth wireless technology 2.0

Frequency range: 1.92 GHz to 1.93 GHz (DECT)
2.402 GHz to 2.48 GHz (Bluetooth)

RF transmission power:

100 mW (MAX)

Power specifications: AC Adapter

Input: 100V-240V AC, 50-60Hz

Output: 12V DC, 1.5A

Use ONLY the power adapter supplied with your unit; using a different adapter may render your unit unusable and voids the warranty.

Power consumption:

Standby: 0.54W

Maximum: 15W

Battery information: Use ONLY rechargeable batteries, NiMH, AA, 1600mAh

Charge time: 8 hours
(from discharged to fully charged)

Note: *The batteries are charging while your phone is powered by the power adapter supplied with your unit.*

Standby time: Approx. 24 hours.

Talk time:


Giant: Approx. 4 hours

Operating conditions: 32 °F – 104 °F,
20 % – 80 % relative air humidity (dry)


PACKAGE CONTENTS

Giant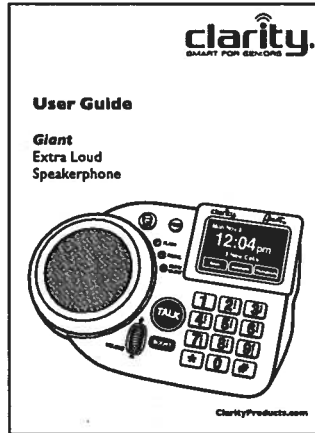

User Guide and
Quick Start Guide

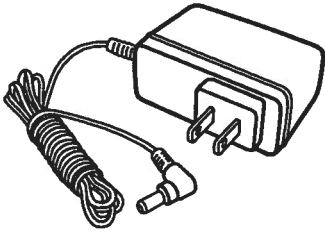AC Power Adapter

Phone cord


Recommended Accessories *(Optional purchase)*

- XLC3.5HSB Expandable Handset (59524.000)
- Voyager Legend Bluetooth Headset (87300-01)

GETTING TO KNOW YOUR SPEAKERPHONE


Please take a moment to review your new product and its features.

This user guide applies to both versions. If your unit does not have an answering machine, ignore references to this feature during the review of this user guide.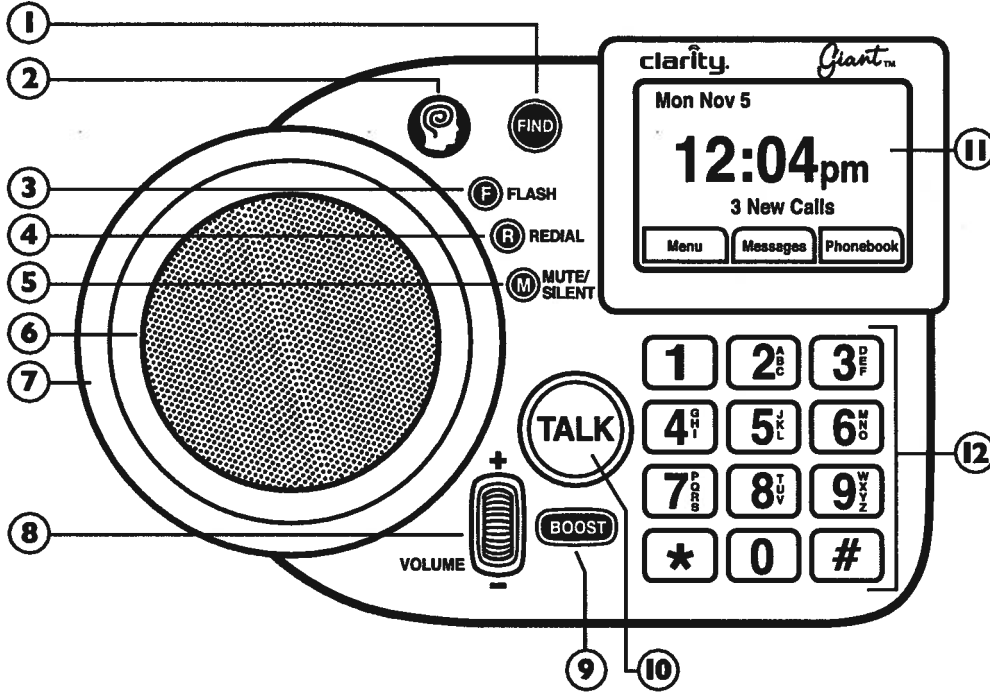

GETTING TO KNOW YOUR SPEAKERPHONE

11	TOUCHSCREEN
12	NUMERIC KEYPAD
13	POWER PORT <ul style="list-style-type: none">• Plug in AC power adapter. <i>(provided)</i>
14	LINE PORT <ul style="list-style-type: none">• Plug in phonecord. <i>(provided)</i>
15	BATTERY COMPARTMENT <ul style="list-style-type: none">• Use 6 AA rechargeable batteries to ensure continuous functionality in case of a power outage. <i>(see battery safety section)</i>

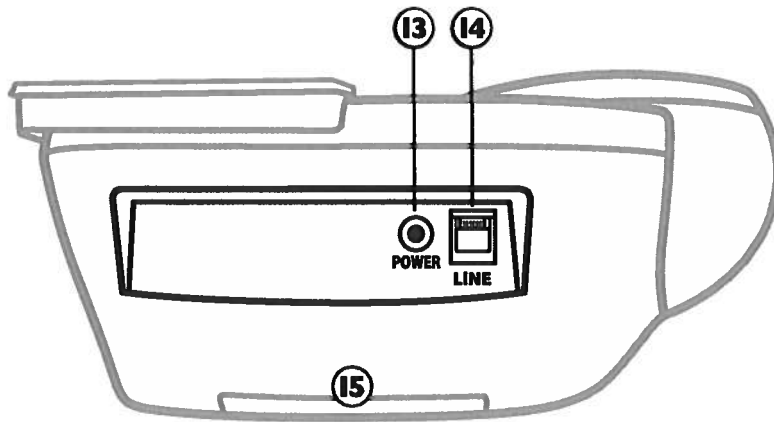
GETTING TO KNOW YOUR SPEAKERPHONE

Menu navigation buttons

(Press Up or Down buttons to scroll the menu listings.)

Functional buttons

Use these buttons to make selections, move backward, save, delete, and other functions.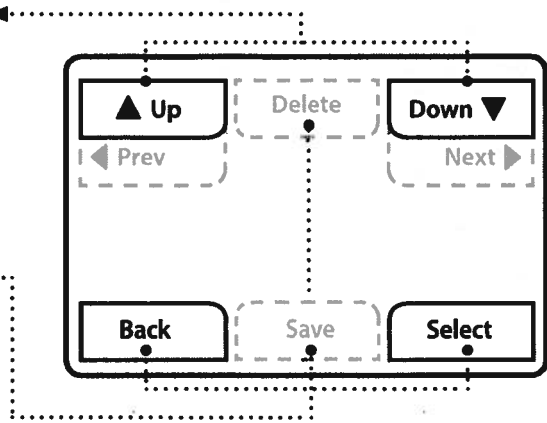

(Figure 1)


Automatic update of date and time

(with CID service)

Note: Must be initially set within the **System Setup Menu, Set Time/Set Date** in order for this feature to work correctly.

Automatic alert status of phone system operations

- Status of Answering Machine (ON/OFF)
- Phone line status (No Line) will display if phone cord not attached.
- Display of New Messages
- Ringer ON/OFF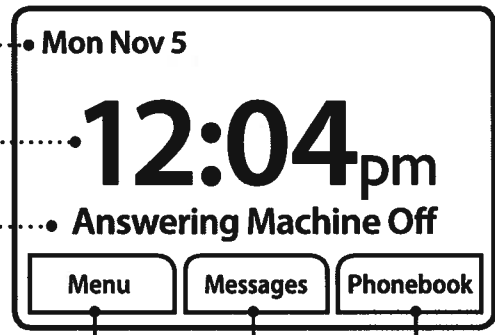

Touch buttons
Access the virtual menu features of the phone.

(Figure 2)

USING YOUR SPEAKERPHONE

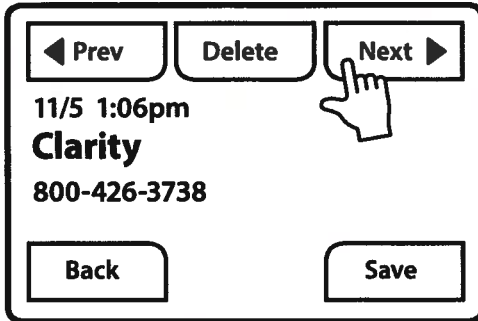
Figure 7

Call timer

Your speakerphone automatically times the duration of every call. During the call the timer (Figure 8) will display the call duration in hours, minutes, and seconds. After the call is made, a “Talk Time” screen will display the total call time.

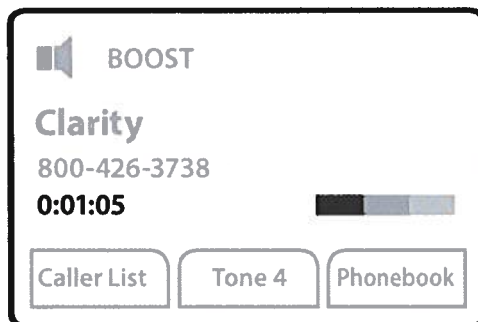
Figure 8

RECEIVING CALLS

Answer a call

When there is an incoming call, the Visual Ringer will blink, and the ring tone will sound. (Figure 9)

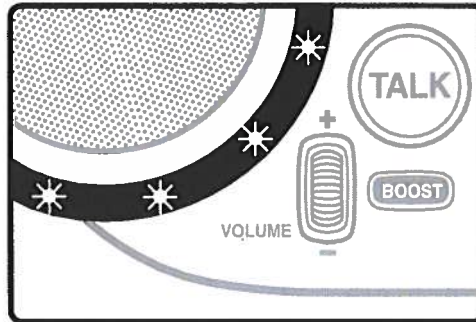
Figure 9

When a call is incoming, the menu screen will display “Ringing”.

Note: To disable the audible ringer, press MUTE/SILENT.

Upon connection, the menu screen will display call information. (Figure 10-11)

Figure 10

USING YOUR SPEAKERPHONE

End a call

During a call you can press the “TALK” button to end the call.

Note: During a call you can press the “Flash” button to switch between calls. However, you must subscribe to Call Waiting as part of your phone service.

FCC REQUIREMENTS AND REGULATIONS

If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this equipment does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

This equipment is hearing aid compatible as defined by the FCC in 47 CFR Section 68.316.

When you hold the phone to your ear, noise might be heard in your Hearing Aid. Some Hearing Aids are not adequately shielded from external RF (radio frequency) energy. If noise occurs, use an optional headset accessory or the speakerphone option (if applicable) when using this phone. Consult with your audiologist or Hearing Aid manufacturer about the availability of Hearing Aids which provide adequate shielding to RF energy commonly emitted by digital devices.

WHEN PROGRAMMING EMERGENCY NUMBERS AND (OR) MAKING TEST CALLS TO EMERGENCY NUMBERS:

1. Remain on the line and briefly explain to the dispatcher the reason for the call.
2. Perform such activities in the off-peak hours, such as early morning or late evenings.

PART 15 OF FCC RULES INFORMATION

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Privacy of communications may not be ensured when using this phone.

CAUTION: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this device.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off

FCC REQUIREMENTS AND REGULATIONS

Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

WARNING: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate.

The Ringer Equivalence Number is an indication of the maximum number of terminals allowed to be connected to a telephone interface. The termination on an interface may consist of any combination of devices subject only to the requirement that the sum of the Ringer Equivalence Numbers of all the devices does not exceed five (5.0).

This Class B digital apparatus complies with Canadian ICES-003. [The term "IC:" before the certification/registration number only signifies that the Industry Canada technical specifications were met.]

PARTY RESPONSIBLE FOR REGULATORY COMPLIANCE:

Clarity, a Division of Plantronics, Inc.
6131 Preservation Drive
Chattanooga, TN 37416
Phone: 800-426-3738

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©2012 Clarity, a Division of Plantronics, Inc. All rights reserved. Clarity, Clarity Logic, Smart for Seniors, and Giant are trademarks or registered trademarks of Plantronics, Inc.